



# FURIX

FVP-1

## Veterinary Pet Syringe Pump

## Features

- ③ 3 Level alarm: High, Medium and Low.
- ③ Sensitive alarm reminder ensures real-time injection safety.
- ③ Comprehensive drug library provides veterinary doctor or nurse with convenient operation and ensures the accuracy of injection drugs.
- ③ The data storage and uploading function facilitates veterinary doctor or nurse to review historical alarm information.
- ③ Automatically identify syringe specifications of 10ml, 20ml, 30ml and 50ml.

## Standard Package

- ③ Check whether there is damage to the surface of the syringe pump, whether to have effective label and power cable.
- ③ Connect the equipment to the power cable and turn it on, and check whether the display screen function and the AC indicator light are working normally.
- ③ Install a 50ml syringe correctly (see syringe installation) correctly, run the syringe pump, and check the interface display under normal operation.
- ③ Execute a quick injection to see whether the display amount and the injection amount are normal.
- ③ Lift the syringe clasp, remove the syringe, and check whether there is a "clasp error" alarm.
- ③ Disconnect the power supply, and the AC indicator light is turned off to check whether the syringe pump is still running normally.
- ③ When the syringe runs out of liquid, there should be "Syringe Emptying" alarm.



Figure 4-2 Schematic diagram of installation completion

### Veterinary Water Warming System

Syringe size	10mL, 20mL, 30ml, 50mL
Clearance/push rate	10ml:0.1-300ml/h, 20ml: 0.1-600ml/h 30ml:0.1-900ml/h, 50ml:0.1-1500ml/h
Infusion mode	Velocity pattern, weight pattern,time pattern
Speed range	0.1-1500ml/h
Velocity increment	0.1ml/h
Working condition	temperature: +5°C~+40°C humidness:30% 80%
Power supply specification	Input voltage: AC220V±22V, Input frequency:50Hz+1Hz
Battery type	11.1V 2000 mAh
Charging time	The charging time in the off state is more than 10h.
Display size	3.2 inches
Product size	336*120*140mm
Product weight	2.0KG

Brand	Syringe specification
WEGO	10ml, 20ml, 30ml, 50ml
DOUBLE-DOVE	10ml, 20ml, 30ml, 50ml



## Furix Trade Venture Limited

Address: 6/F MEGA CUBE, WANG KWONG ROAD, KOWLOON, HONG KONG,  
Mob:+852 9579 0472 E-mail: vet.furix@gmail.com, sales@myfurix.com  
Tel:+852 9579 0472 www.myfurix.com

# FURIX